630-011

<b>SE Automatisk blodtrycksmätare för handled</b> .....	2
<b>DK Automatisk blodtryksmåler til håndled</b> .....	9
<b>NO Automatisk blodtryksmåler til håndleddet</b> .....	16
<b>FI Automaattinen verenpainemittari</b> .....	23
<b>UK Automatic Blood Pressure Monitor</b> .....	30
<b>DE Vollautomatisches Blutdruckmeßgerät</b> .....	37
<b>PL Automacyjny ciśnieniomierz do mierzenia ciśnienia w nadgarstku</b> .....	45
<b>RU Аппарат для автоматического измерения кровяного давления на запясть</b> .....	52

# SE

## INTRODUKTION

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## INTRODUCTION

To get the best out of your blood pressure monitor, please read through these instructions carefully before using it for the first time. Pay particular attention to the safety measures. We also recommend that you keep the instructions for future reference, so that you can remind yourself of the functions of the blood pressure monitor.

## IMPORTANT SAFETY MEASURES

- Use the blood pressure monitor for its intended purpose only.
- Do not use the blood pressure monitor if it is not functioning properly, or if it has been damaged by a heavy impact or similar.
- The armband must not be stretched, twisted or bent.
- Do not use the blood pressure monitor on babies or others who cannot communicate if the armband is too tight, etc.
- The blood pressure monitor must not be immersed in water or any other liquid.
- Do not use the blood pressure monitor in the vicinity of mobile phones or microwave ovens as these emit electromagnetic radiation that can affect the blood pressure monitor.
- Do not expose the monitor to direct sunlight, high temperatures, humidity, noise or corrosive substances.

- Never try to repair the blood pressure monitor yourself. If it needs to be repaired, it should be taken to an authorised service centre. If the appliance is subjected to unauthorised repairs, the guarantee will become void. Contact your retailer if the repairs fall under the terms of the guarantee.
- The blood pressure monitor must not be used to make medical diagnoses but should only be used for reference. Consult a doctor if your blood pressure readings seem abnormal.
- Always follow the procedure specified in these user instructions when taking measurements.

## Batteries

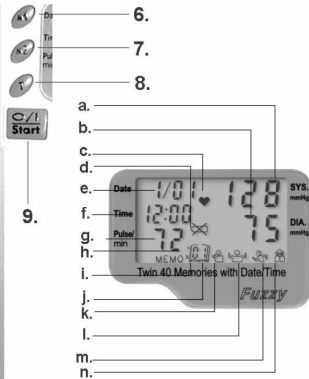
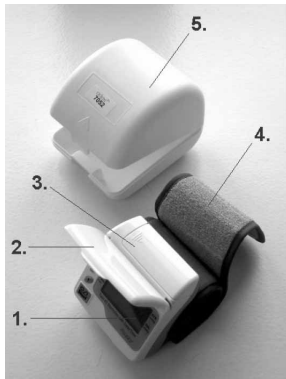
- Remove the batteries from the blood pressure monitor if it is not to be used for a prolonged period.
- Change the batteries immediately once they are dead.
- Never use different types of batteries, or use a new and an old battery at the same time.
- Ensure the batteries are correctly polarised when putting them in the blood pressure monitor.

## KEY TO THE BLOOD PRESSURE MONITOR'S PARTS

1. Display
2. Protective cover
3. Battery cover
4. Armband
5. Storage box
6. M1/up button
7. M2/down button
8. T button
9. I/O/Start button

### Symbols on the display

- a. Systolic pressure
- b. Diastolic pressure
- c. Pulse symbol
- d. Battery symbol
- e. Month and date
- f. Time/year
- g. Pulse
- h. Memory 1
- i. Memory 2
- j. Measurement number
- k. Hypotension
- l. Normal
- m. High normal
- n. Hypertension

## ABOUT BLOOD PRESSURE

What is blood pressure?

When the heart pumps blood around the body, the blood exerts pressure on the walls of the blood vessels. This is known as blood pressure.

- Blood pressure is different in different parts of the body and depends on the pulse, how elastic the blood vessels are and how thick the blood is.
- The pressure that can be measured when the heart pumps out into the body is known as systolic blood pressure. The pressure that can be measured when blood runs back from the body is known as diastolic blood pressure.
- The two types of blood pressure are usually given together, with the systolic blood pressure always given first. If blood pressure is given as "120/70" (expressed as "120 over 70"), this means that the systolic blood pressure is 120, and the diastolic 70.
- The measurement unit for blood pressure is mmHg (millimetres of mercury).

### Why does blood pressure vary?

Blood pressure varies throughout the day and over a person's life. Many different factors affect your blood pressure, something you should be aware of when taking your blood pressure. For instance, be aware of the following:

- Blood pressure is higher during the day than at night.
- Blood pressure increases slightly after you have just eaten. A lot of salt in food can also increase blood pressure.
- Light exercise (e.g. walking and jogging) causes the systolic blood pressure to rise slightly, while hard exercise

will cause it to rise dramatically. However, it will fall again when the body relaxes. A lack of exercise raises blood pressure permanently.

- Blood pressure can increase if you are excited, stressed or tense.
- Sudden climatic changes can cause blood pressure to rise.
- The systolic pressure increases with age.
- Women have a greater risk of increased blood pressure after the menopause.
- Insufficient sleep, smoking, obesity, alcohol and certain illnesses cause blood pressure to rise.
- Bathing and toilet visits may cause blood pressure to rise.

### When is my blood pressure too high or too low?

High blood pressure is also called hypertension. Low blood pressure is also called hypotension. The diagram below shows when your blood pressure is too high or too low.

Range	Systolic	Diastolic
Hypertension	> 140 mmHg	> 90 mmHg
High normal	130-139 mmHg	85-89 mmHg
Normal	91-129 mmHg	51-84 mmHg
Hypotension	< 90 mmHg	< 50 mmHg

### Why should I measure my blood pressure?

Blood pressure naturally increases as you get older, but high blood pressure can also be the first sign of illness, being overweight or of the arteries beginning to stiffen, which increases the risk of blood clots, among other things. It is therefore a good idea to keep an eye on your blood pressure and consult a doctor if it seems unusually high or low.

**Important! The blood pressure monitor must not be used to make medical diagnoses but should only be used for reference. Consult a doctor if your blood pressure readings seem abnormal. Always follow the procedure specified in these user instructions when taking measurements.**

## **PREPARING THE BLOOD PRESSURE MONITOR**

### **Inserting the batteries**

- Remove the battery cover (3) by pushing it to one side.
- Insert two AAA batteries. Check that the batteries are in the right way round. Replace the cover.
- Change the batteries when the battery symbol (d) is shown in the display (1).
- Please note! If the batteries are changed in less than 1 minute, you will not need to reset the clock and any saved blood pressure readings (see below) will remain in the blood pressure monitor's memory. If it takes more than 1 minute to change the batteries, the display will show all symbols for a moment. The blood pressure emits a short beep and is reset. Any saved blood pressure readings are deleted. The display then shows the reset time.

### **Setting the clock**

- Press and hold the T button (8) until the year number flashes on the display. Use the M1 (6) and M2 (7) buttons to set the year number, and press the T button when it is correctly set.
- The month number now flashes on the display. (Please note! The blood pressure monitor shows the month before the date). Enter the month number in the same way as the year number, and press the T button when it is correctly set.

- The date, hour number and minute number now flash in turn and are set in the same way as described above.
- The display now shows the date (e) and time (f). If you press the T button briefly, the year number is shown briefly instead of the time.

## **OPERATING THE BLOOD PRESSURE MONITOR**

### **Preparing for a blood pressure reading**

- Sit or lie down for 5-10 minutes before the reading.
- Avoid eating, smoking, exercising, bathing or drinking alcohol, coffee or tea before taking the reading.
- Do not go to the toilet immediately before the reading.
- Do not take a blood pressure reading if you are excited, stressed or tense.
- Relax during the reading and do not talk.

### **Blood pressure reading**

- Place the armband (4) on one of your wrists so that the blood pressure monitor is on the underside of your arm and tighten it using the Velcro strap. It does not matter if you take the reading on your left or right wrist. The armband should be taut but should not pinch. The side of the armband facing your hand must be 0.5-1 cm from the hand.
- Place the arm to be measured on a pillow on the table with the palm facing upwards and the blood pressure monitor at roughly the same height as your heart.
- Press the I/O/Start button (9).

- A beep is heard and the number 0 flashes on the display. Shortly afterwards, the armband is pumped up to 185 mmHg, while the pressure is continuously shown on the display. The blood pressure monitor buzzes while the armband is being pumped up.
- When the armband is pumped up to 185 mmHg, the displayed figure falls again and the blood pressure reading starts. The blood pressure monitor emits a number of beeps while the reading is being taken. The pulse is also measured, and the pulse symbol (c) is shown on the display. The armband is automatically pumped up if the pressure is too low.
- Once the blood pressure reading has been taken, a long beep is heard and the display shows the systolic pressure (a), the diastolic pressure (b), the pulse (g) and the time. There is also a symbol for the range the measured blood pressure falls under: hypotension (k), normal (l), high normal (m) or hypertension (n). The results are shown on the display for around 3 minutes and then deleted. You can also delete them by pressing the I/O/Start button again.
- If you need to take several readings, you should wait 5-10 minutes between each reading.
- Please note! An individual reading says nothing about your state of health. You can only get an idea of this after a number of readings.

### **Storing blood pressure readings**

- The blood pressure monitor can store 80 blood pressure readings in two groups of 40 readings. Once you have taken a blood pressure reading and the results are shown on the display, you can press the M1 button to save the reading in group 1 or M2 to save the reading in group 2.
- The display shows MEMO, a beep is heard and the reading is saved.

### **Displaying saved blood pressure readings**

- Press the M1 button to display the saved blood pressure readings in group 1 or the M2 button to display the saved blood pressure readings in group 2.
- The first reading in the selected group is displayed for approximately 4 seconds. The display shows all the data for the reading plus the reading number (j).
- Press M1/M2 repeatedly to display the subsequent readings in the selected group. Press and hold the button to scroll through the readings. The reading number is always displayed.
- The blood pressure monitor automatically changes back to the time display. If required, press the T button to change back more rapidly.

## ERROR MESSAGES

The blood pressure monitor registers any problems and shows the following error messages on the display:

<b>Error message</b>	<b>Meaning</b>	<b>Cause</b>	<b>Solution</b>
ER-1	Error during pumping	<ol style="list-style-type: none"><li>1. The armband is not correctly attached.</li><li>2. The armband is broken.</li><li>3. The pump or valves are defective.</li></ol>	<ol style="list-style-type: none"><li>1. Attach the armband correctly.</li><li>2. Contact the retailer.</li><li>3. Contact the retailer.</li></ol>
ER-2	Disturbances or noise during readings	<ol style="list-style-type: none"><li>1. The user moves or speaks during the reading.</li><li>2. The user is too nervous.</li></ol>	<ol style="list-style-type: none"><li>1. The user must sit still and avoid talking during the reading.</li><li>2. Have a blood pressure reading taken by a doctor.</li></ol>
ER-3	Error while emptying the armband	<ol style="list-style-type: none"><li>1. The valves are defective.</li><li>2. The armband is not correctly attached.</li></ol>	<ol style="list-style-type: none"><li>1. Contact the retailer.</li><li>2. Attach the armband correctly.</li></ol>
ER-4	Abnormal reading	The user may have a heart problem.	Consult a doctor.
HI	The pressure in the armband is too high.	The air tube is blocked, the pump mechanism is defective or the pressure monitor is defective.	Try to take another reading, or contact the retailer.
Battery symbol	Flat batteries	The batteries are almost dead.	Replace the batteries.

## **STORAGE**

When the blood pressure monitor is not in use, it should be stored in the accompanying storage box (5). Store the blood pressure monitor in a dry, cool place. Remove the batteries from the blood pressure monitor if it is not to be used for a prolonged period.

## **CLEANING**

When cleaning the blood pressure monitor, you should pay attention to the following points:

- Use a soft, dry cloth to clean the blood pressure monitor. The armband can be cleaned using a damp cloth with a little detergent.
- You must not use strong, corrosive or abrasive cleaning agents for cleaning the surface of the blood pressure monitor, as these may scratch the surface.

## **ENVIRONMENTAL TIPS**

Once any electronic product is no longer functional, it should be disposed of in such a way as to cause minimum environmental impact, in accordance with the regulations of your local authority. In most cases you can take such products to your local recycling station.

## **GUARANTEE TERMS**

The guarantee does not apply:

- if the above instructions are not followed
- if the appliance has been interfered with
- if the appliance has been mishandled, subjected to rough treatment, or has suffered any other form of damage

Due to the constant development of our products in terms of function and design, we reserve the right to make changes to the product without prior warning.

## **IMPORTER**

Adexi Group

We cannot be held responsible for any printing errors.

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