
GEBRAUCHSANLEITUNG
INSTRUCTIONS FOR USE
NOTICE D'EMPLOI
ISTRUZIONI PER L'USO
INSTRUCCIONES PARA EL USO
GEBRUIKSAANWIJZING

SET 250
SET 500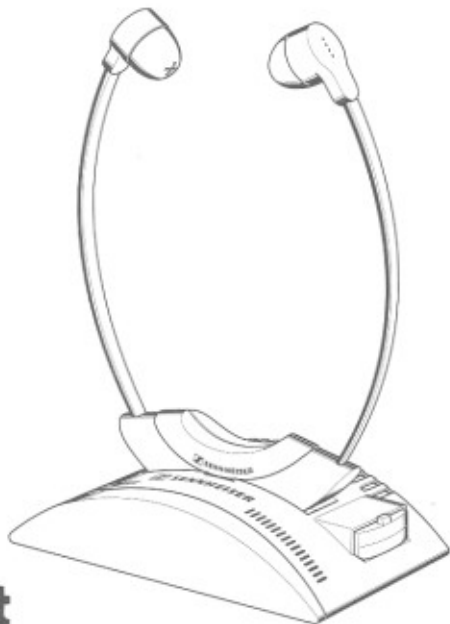

inraport

Gebrauchsanleitung	3
Instructions for use	13
Notice d'emploi	23
Istruzioni per l'uso	33
Instrucciones para el uso	43
gebruiksaanwijzing	53

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THANK YOU FOR CHOOSING SENNHEISER!

We have designed this product to give you reliable operation over many years.

Please take a few moments to read these instructions carefully, as we want you to enjoy your new Sennheiser product quickly and to the full.

PRODUCT VARIANTS

▶ SET 250

Infra-red mono sound transmission. This system consists of the TI 250 infra-red transmitter and the RI 250 stethoset receiver. A typical area of application is cordless sound transmission from mono television sets.

▶ SET 500

Infra-red stereo sound transmission. This system consists of the TI 500 infra-red transmitter and the RI 500 stethoset receiver. A typical area of application is cordless sound transmission from stereo television sets or hi-fi systems. Other applications are also conceivable, such as connection to CD players, multimedia computers, telephone etc.

This manual describes both product variants. There is no difference in operation between the mono and stereo systems.

SENNHEISER: ADDITIONAL INFORMATION

- ▶ Your local Sennheiser dealer will be happy to provide you with information on the extensive range of Sennheiser products. He has a variety of printed materials on Sennheiser products.
- ▶ For up-to-date information on Sennheiser products, you can also visit the Sennheiser WWW site at "<http://www.sennheiser.com>".

ADVANTAGES

- ▶ With cordless sound transmission via a stethoset receiver, you can be independent of cables. Only you can hear the television sound, and you can decide the volume yourself without disturbing others!
- ▶ According to present scientific knowledge, sound transmission by infra-red light poses no risk to health whatsoever.
- ▶ High reception reliability due to the carrier frequencies of 2.3 MHz and 2.8 MHz. Fluorescent tubes and digital controls do not use this frequency range.
- ▶ Lightweight and extremely comfortable to wear - the receiver weighs only 45 g, including the rechargeable accuplug.
- ▶ Replaceable ear-cushions of silicone rubber which are kind to the skin.
- ▶ Mains connection via plug-in mains unit. The transmitter thus remains small and unobtrusive. If you take your system to a country with a different mains voltage, all you need to do is purchase the suitable plug-in mains unit.

SENNHEISER TECHNOLOGY AND SENNHEISER SERVICE ARE FUTURE-ORIENTED

Even after many years, you will still have unlimited pleasure from your infra-red sound transmission system. You can buy replacements for wearing parts such as ear-cushions and accuplugs for many years to come. And in the unlikely event that any repairs should be necessary, Sennheiser Service Departments worldwide give moderately priced, fast and competent repairs.

ENVIRONMENT AND HEALTH

Devices powered by rechargeable accuplugs are kind to the environment. This energy source can be recharged many times. Conventional batteries, on the other hand, must be disposed of as special waste after they have been used just once. Sennheiser uses environmentally friendly NiMh rechargeable accuplugs which contain no heavy metals. If an accuplug should be defective, it is then recycled - the dealer who sold it to you will take it back and will also supply you with a new, original Sennheiser accuplug. (Recycling methods differ in different countries).


THIS PACKING UNIT CONTAINS ...

... several individual products. Your new infra-red sound transmission system consists of five components:

- ① The **TI 250 / TI 500 transmitter** converts the sound signal from the headphone socket of your television set into invisible infra-red light and radiates this light into the room.
Integrated in this transmitter is a twin charging unit in which a BA 151 accuplug in the RI 250 / RI 500 receiver and an additional BA 151 accuplug can be charged.
- ② The transmitter (with the twin charging unit) is powered via a **plug-in mains unit**.
- ③ You can connect your television set or hi-fi system with the supplied **cable**. The 3.5 mm mini jack plug fits most headphone sockets. An **adaptor** for 1/4" (6.3 mm) headphone sockets is also included in the supply schedule.
- ④ The **RI 250 / RI 500 stethoset receiver** receives the infra-red light and converts it back into an audible sound signal. The receiver and headphones are integrated in a single stethoset receiver. This means that no troublesome cables are needed.
- ⑤ **BA 151 accuplug** for powering the stethoset receiver.
- ⑥ **Replacement ear-cushions** for the stethoset receiver.
- ⑦ Two **sealing tapes** for sound adjustment.

Please also note any enclosed documents which inform you of alterations or country-specific variants. Such documents are supplements to this instruction manual.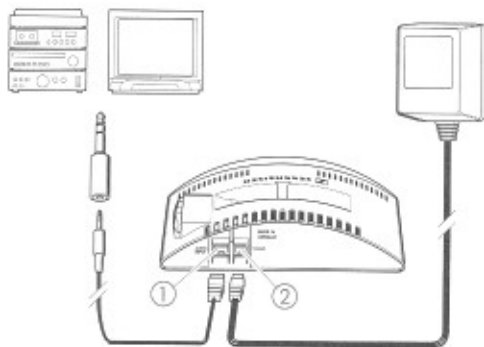

SETTING UP THE TRANSMITTER

Place the transmitter near your television set or hi-fi system.

PUTTING THE TRANSMITTER INTO OPERATION


- ▶ Use the supplied cable to connect socket ① on the TI 250 / TI 500 transmitter with the headphone socket on your television set or hi-fi system.

NOTE

If your television set does not have a headphone socket, the sound signal can also be taken from the SCART socket (video recorder socket). Please order the suitable Sennheiser adaptor (Cat. no. 47874) from your Sennheiser dealer.

Only for the TI 250 mono transmitter: For television sets without any suitable headphone sockets, the sound can also be taken directly from the loudspeaker by using the Sennheiser MKE 100 TV microphone (Cat. no. 03354). Connection to socket ①. (This is supplied instead of the headphone socket cable in the UK version).

- ▶ Connect the supplied plug-in mains unit to socket ② on the TI 250 / TI 500 transmitter, then plug the mains unit into the mains.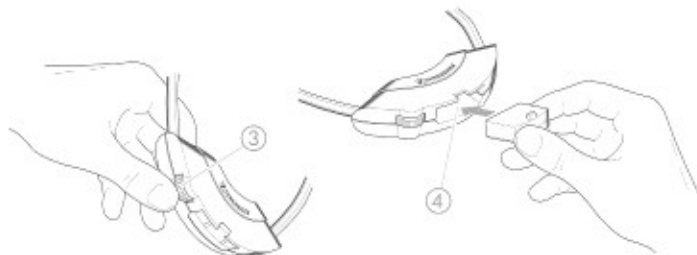

LEVEL CONTROL OF THE TRANSMITTER

Adjust the volume of your hi-fi system or television set to a middle position. The TI 250 / TI 500 transmitter features an automatic modulation control which prevents overmodulation.

PUTTING THE RECEIVER INTO OPERATION

Insert the BA 151 accuplug into compartment ④ in the receiver.

Take special note of the instructions on accuplug operation on pages 20/21 of this manual!


(NB: The BA 151 accuplug must be charged for 24 hours before first use!)

SWITCHING ON THE RECEIVER

Use control ③ to switch on the receiver and adjust the volume. An efficient volume compression system avoids the necessity of frequent volume adjustments. It reproduces quiet sounds slightly louder and loud sounds slightly quieter.

ADJUSTING YOUR HI-FI SYSTEM

Read the instruction manual of your hi-fi system or television set to find out whether the volume control also works for the headphone socket, or whether this socket has a separate volume control. The manual will also tell you how to turn off the main loudspeakers.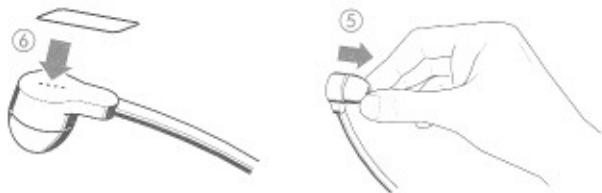

ADJUSTING THE BASS ATTENUATION

You can use the two supplied sealing tapes to seal the small holes in the headphone systems ⑤. The more holes you seal with the sealing tapes, the greater is the bass attenuation. This increases the intelligibility without increasing the volume - which is an advantage for persons with a slight hearing impairment.

REPLACING THE EAR-CUSHIONS


The ear-cushions can easily be removed from the headphones and cleaned with warm soapy water ⑥. From time to time, however, the ear-cushions should be replaced for reasons of hygiene. Replacement ear-cushions are available with different degrees of softness:

- ▶ Black ear-cushions, firmer version Cat. no. 37080
- ▶ Transparent ear-cushions, softer version Cat. no. 40949

Please order your ear-cushions from your Sennheiser dealer.

VOLUME UP? – NO!

When people use headphones, they tend to choose a higher volume than with loudspeakers. Listening at high volume levels for long periods can lead to permanent hearing defects. Please protect your hearing, Sennheiser headphones have an excellent sound quality even at low volumes.

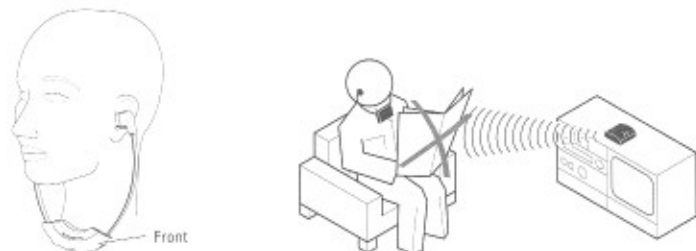FREE LINE OF SIGHT

Infra-red light from the TI 250 / TI 500 transmitter is transmitted in the same way as the light from a light bulb. Do not obstruct the path of the light between the transmitter and the receiver, and stay in the room where the transmission is taking place! Make sure that you wear the stethoset receiver the right way round, with the „Sennheiser“ logo at the front.

There is a simple rule. If you can see the transmitter from where you are, transmission can take place without any interference.

ADJUSTMENT FOR HEARING IMPAIRMENT

Is your hearing capacity different for each ear? You can use the balance control ⑦ to adjust your RI 250 / RI 500 stethoset receiver to your individual needs.

The balance control ⑦ is recessed and is adjusted with a small screwdriver. This adjustment is only changed occasionally - and it is best done by the hearing aid dealer who sold you your SET 250 / SET 500.

CHARGING THE ACCUPLUG / CHARGE INDICATION

- ▶ To charge the accuplug, insert the receiver containing the accuplug into the storage and charging compartment ⑧. The charging process is indicated by the LED ⑩. Switch off the receiver before charging the accuplug!
- ▶ A second BA 151 accuplug can be charged in the charging compartment ⑨. This charging process is indicated by the LED ⑪.
- ▶ When the accuplug becomes weak (noise interference and sound distortions are sure signs of a weak accuplug), switch off the receiver and recharge the accuplug. (If necessary, use a second accuplug to listen while the first accuplug is being recharged.)
- ▶ Recharge the accuplug immediately when it is flat. If the accuplug is flat but is left in the switched on receiver, this reduces the service life of the accuplug („total discharge“).
- ▶ As far as possible, use the full capacity of an accuplug before you recharge it. Frequent short discharge and recharge cycles prematurely reduce its charging capacity.
- ▶ **Charge the accuplug for 24 hours before you use it for the first time!**

NOTE

- ▶ Charging in the charging compartments ⑧ and ⑨ works even when the transmitter is switched off as long as it remains connected to the mains!
- ▶ **The transmitter is switched off when you recharge the receiver containing the accuplug in the charging compartment ③!**

RECHARGEABLE ACCUPLUGS HAVE A „CHARGING CYCLE“

Depending on how long your receiver has been in use, you must also recharge the BA 151 accuplug for a certain time. A simple rule of thumb will help you to decide what time is necessary:

$$\text{Operating time} \times 3 = \text{recharging time}$$

Example:

1 hour listening
3 hours recharging

2 hours listening
6 hours recharging


3 hours listening
9 hours recharging


OPERATION WITH A SECOND ACCUPLUG

By using a second accuplug, you can increase the listening time with your receiver. The second accuplug is always inserted in the charging compartment ⑨ of the transmitter to ensure that it is ready and fully charged when it is needed.

- ▶ If you use your system with only one accuplug, you must stop after about five hours and recharge the accuplug for about 15 hours in the charging compartment ⑨ of the transmitter.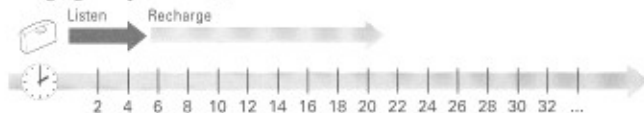

- ▶ With a second accuplug, you can listen for more than twice the time. Simply place the discharged first accuplug back into the charging compartment. After the (partial) recharge, there is an additional (partial) amount of power available:

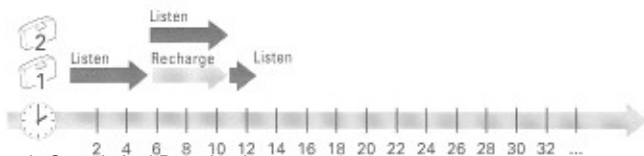SAFETY INSTRUCTIONS

- ▶ Use the transmitter only with the supplied Sennheiser plug-in mains unit. Always disconnect the mains connector when you wish to change connections or move the device to a different place.
- ▶ Never open electronic devices! This must only be done by authorised personnel and is all the more important for current-carrying units.
- ▶ Keep the system away from central heating radiators and electric heaters. Never expose it to direct sunlight.
- ▶ Use the system in dry rooms only.
- ▶ Use a damp cloth for cleaning the system. Do not use any solvents.

TECHNICAL DATA

	SET 250	SET 500
Modulation	FM, mono	FM, stereo
Carrier frequencies	2.3 MHz	2.3 MHz / 2.8 MHz
Audio frequency response	18–18,000 Hz	18–18,000 Hz
THD	1 %	1 %
Signal-to-noise ratio	typ. 70 dB	typ. 64 dB

RECEIVER

	RI 250	RI 500
Operating time with one BA 151accuplug	approx. 6 h	approx. 5 h
Weight with / without BA 151 accuplug	45 g / 35 g	45 g / 35 g
Max. SPL	124 dBA	124 dBA

TRANSMITTER

	TI 250	TI 500
Power consumption	approx. 7 VA	approx. 7 VA
Power supply plug-in mains unit:		
European version	NG 12-DC	NT 20
USA version	NG 12-120	NT 20-120
UK version	NG 12-UK	NT 20-UK
Weight	approx. 170 g	approx. 180 g

ACCESSORIES

BA 151 accuplug	Cat. no. 04146
RI 250 receiver	Cat. no. 04373
RI 500 receiver	Cat. no. 04376
Replacement ear-cushions, black (firmer version)	Cat. no. 37080
Replacement ear-cushions, transparent (softer version)	Cat. no. 40949
MKE 100 TV microphone (for TI 250 only)	Cat. no. 03354

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Sennheiser Irland GmbH